

## HOME CARE MASKS & ACCESSORIES

A Solution For Each Patient



NASAL MASKS

FULL FACE MASKS

MINIMAL CONTACT MASKS

ACCESSORIES

MASK FITTING

DEVICES

# Which mask for which patient?

All CPAP & Bi-Level Patients

If Mouth Breathers



**ComfortGel Full**



**ComfortGel**



If looking for a minimal contact solution

Pillows cushion



**OptiLife**

Cradle Cushion



**OptiLife**










Nasal cushion




**ComfortLite 2**

2-in-1 solution

# Patient Interface Line Overview

	NASAL MASKS				FULL FACE MASKS		MINIMAL CONTACT MASKS		
	<i>ComfortGel</i>	<i>ProfileLite</i>	<i>ComfortFusion</i>	<i>ComfortClassic</i>	<i>ComfortGel Full</i>	<i>ComfortFull 2</i>	<i>ComfortLite 2</i>	<i>OptiLife 2-in-1 solution</i>	
									
Sizes	P, S, M, L	P, S, SM, M, ML, L, LN, Child	S, M, FitPack (S&M) DuoPacks (S&S, M&M)	S, M	S, M, L	S, M, L	P, S, M, L	P, S, M, L	S, M, L, LN
SE version (no exhalation port)	Yes	Yes	Yes	No	Yes	Yes	No	No	No
Replaceable cushion	Yes	No	Yes	No	Yes	No	Yes	Yes	Yes

**Forehead Support  
and Silicone Forehead Pad**  
1018401  
**Silicone Forehead Pad Only**  
1009722 (4/Pk)

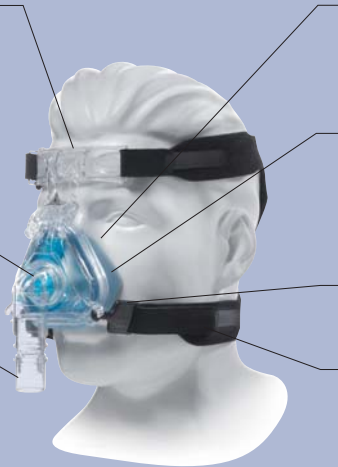

**Silicone Comfort Flap**  
1009052 (P)  
1009053 (S)  
1009054 (M)  
1009109 (L)

**Blue Gel  
Inner Cushion**  
1009048 (P)  
1009049 (S)  
1009050 (M)  
1009051 (L)

**Ball-and-Socket  
Headgear Quickclips**  
1009723

**Premium Headgear**  
1033678

**Exhalation  
Elbow Kit**  
1010646

**Swivel**  
7041

# ComfortGel




The unique gel technology to optimize comfort and drive long-term therapy

Size	P	S	M	L
W/ exh.	1009044	1009045	1009046	1009047
W/O exh.	1010642	1010643	1010644	1010645



W/O exh. port

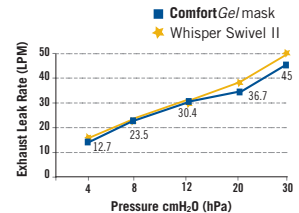




# Profile*Lite*

The original gel mask without silicone and fewer adjustment features

Size	P	S	MS	M	MW	L	LN	CHILD
W/ exh.	1004113	1004114	1004115	1004116	1004117	1004118	1004119	1008829
W/O exh.	1006314	1006315	1006316	1006317	1006318	1006319	1006320	




W/O exh. port

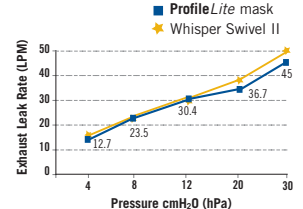

- Gel Retrofit Head Spacer**  
1004292 (S)  
1004291 (L)  
1008869 (XL)
- Exhalation Elbow Kit**  
1004395
- Port Cap**  
152033 (10/Pk)
- Swivel**  
7041
- Deluxe Headgear**  
1002801

**Profile*Lite* Child**  
1008829

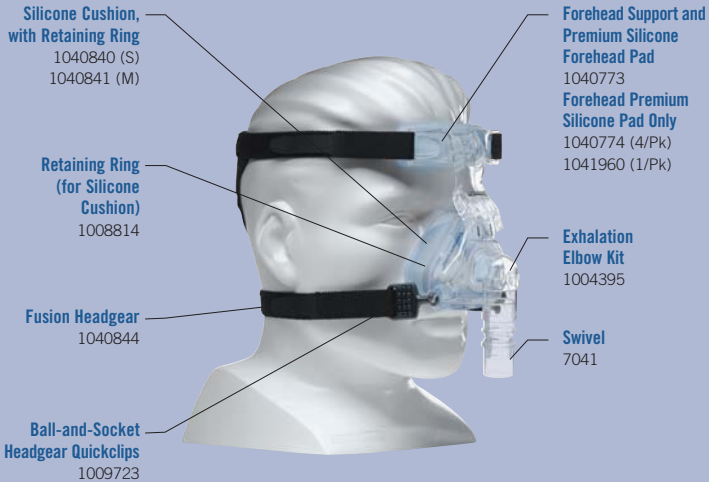


## ComfortFusion


The most effective replacement mask for silicone nasal mask users

Size	S	M	FitPack S&M	DuoPack S&S	DuoPack M&M
W/ exh.	1042211	1042212	1040838	1042213	1042214
W/O exh.	1042215	1042216			



W/O exh. port

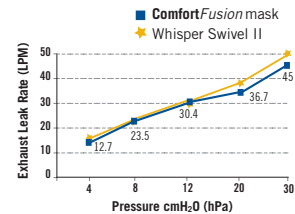

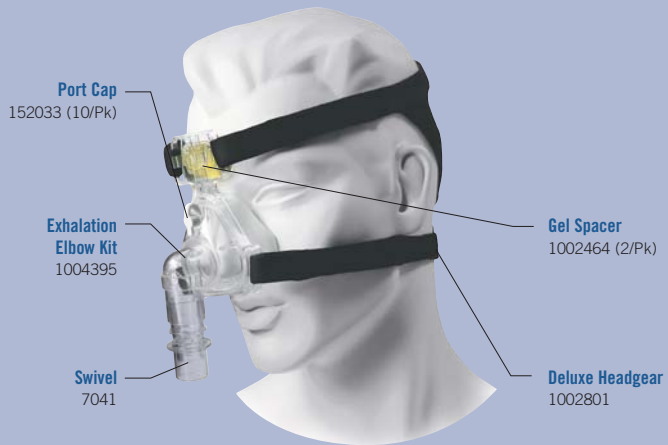

## ComfortClassic


The simplest silicone mask with fewest adjustments

Size	S	M
W/ exh.	1007967	1007968



302209

Mask Swivel without Exhalation Port to be connected when no port on the mask is required.

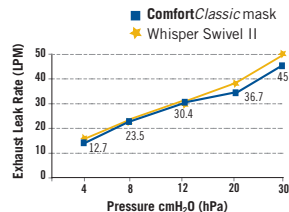

**ComfortGel Full  
SST Flap Only**  
1040795 (S)  
1040796 (M)  
1040797 (L)

**ComfortGel Full  
Cushion Only**  
1040798 (S)  
1040799 (M)  
1040800 (L)

**ComfortGel Full Flap  
and Gel Cushion**  
1044861 (S)  
1044862 (M)  
1044863 (L)

**ComfortGel Full  
Premium Headgear**  
1040138



**Premium Silicone  
Forehead Pad**  
1040114 (4/Pk)

**Port Cap**  
1010872 (5/Pk)

**Entrainment  
Valve Swivel**  
1049398

**ComfortGel Full  
Headgear Quickclips**  
1040139

# ComfortGel Full




The recognised gel technology now available for mouth breathers. Immediate comfort and long-term compliance

Size	S	M	L
W/ exh.	1040140	1040141	1040142
W/O exh.	1043550	1043551	1043552



W/O exh. port

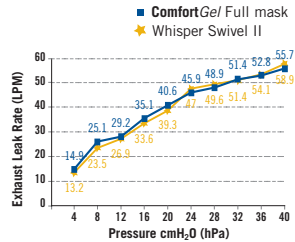




## ComfortFull 2


The basic silicone fullface mask for mouth breathers

Size	S	M	L
W/ exh.	1004881	1004873	1004951
W/O exh.	1023747	1024261	1024262



W/O exh. port

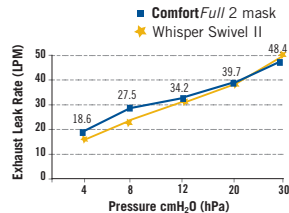

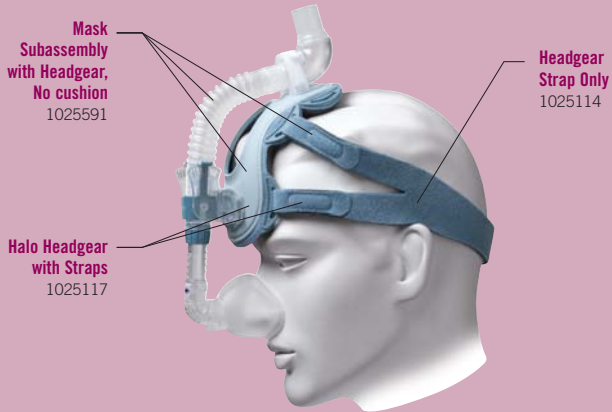

Mask  
Subassembly  
with Headgear,  
No cushion  
1025591

Headgear  
Strap Only  
1025114

Halo Headgear  
with Straps  
1025117



**Pillows Cushion**

- 1030502 (P)
- 1030503 (S)
- 1030504 (M)
- 1030505 (L)



**Simple Cushion**

- 1021835 (P)
- 1021830 (S)
- 1021823 (M)
- 1021821 (L)



**Direct Seal**

- 1021802 (1)
- 1021806 (2)
- 1021807 (3)
- 1021815 (4)
- 1021816 (5)
- 1021817 (6)

# ComfortLite 2


The minimal contact alternative  
to nasal masks

**One cushion-type Packs**

	Simple cushion	Pillows	Direct Seal
S, M, L	1025171	1030501	1025170

**Combo Packs**

	Simple cushion & Pillows	Simple cushion & Direct Seal
S & M	1030499	1025168
M & L	1030500	1025169

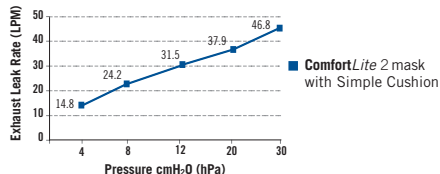

Headgear including  
Chin Support Band  
1036850



OptiLife Pillows Cushion  
1036838 (P)  
1036839 (S)  
1036840 (M)  
1036841 (L)

OptiLife Subassembly  
*No cushion,  
with headgear*  
1036836  
*No cushion,  
no headgear*  
1036837

**New Cushion Option**

CradleCuhion  
1036846 (S)  
1036847 (M)  
1036848 (LN)  
1036849 (L)



# OptiLife

Maximum comfort  
in a 2-in-1 minimal contact mask solution

**One cushion-type Pack**


Each cushion size individually wrapped inside carrying bag

P, S, M, L

1036832

**Combo 2-in-1 Pack**

	Pillows & Cradle	Pack Content
S & M	1036833	Pillows P, S, M Cradle S, M
M & L	1036834	Pillows M, L Cradle M, L, LN

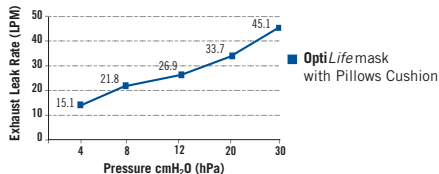

# Accessories

## Headgears

Part Number	Headgears
1033678	Premium Headgear with EZ Peel Tabs ( <i>fits perfectly to any type of nasal or full face mask</i> )
1040138	Blue Premium Headgear with EZ Peel Tabs ( <i>Alternative headgear for ComfortGel Full mask</i> )
1040844	Fusion Headgear ( <i>Alternative Headgear for ComfortFusion mask</i> )
1002801	Deluxe Headgear ( <i>Alternative Headgear for ProfileLite and ComfortClassic masks</i> )
1014334	ComfortHeadgear ( <i>Alternative Headgear for ComfortFull2 mask</i> )
1002285	SimpleStrap Headgear



1033678



1040138



1002801






1014334



1002285

## Softcaps

Part Number	Softcaps (only fit to ProfileLite, ComfortClassic, and Contour masks)		
<b>Mesh Softcap</b>			
1001632	Mesh Softcap - Child	P	
1002806	Mesh Softcap - Medium size	M	
1002807	Mesh Softcap - Large size	L	
1001634	Mesh Softcap - X-Large size	XL	
<b>White Lycra Softcap</b>			
1001624	White Softcap - Child	P	
1001625	White Softcap - Medium size	M	
1001626	White Softcap - Large size	L	
1001627	White Softcap - X-Large size	XL	
<b>Blue Lycra Softcap</b>			
1001628	Blue Softcap - Child	P	
1001629	Blue Softcap - Medium size	M	
1001630	Blue Softcap - Large size	L	
1001631	Blue Softcap - X-Large size	XL	

## Chin Straps

Part Number	Chin Straps
1012911	Premium Chin Strap
1006085	Deluxe Chin Strap
302175	Chin Strap



1012911



1006085



302175

## Reusable Circuits

Part Number	Reusable circuits
622038	Reusable Flexible Tubing, Grey (1,83 m)
1032907	Performance Tubing, White (1,83 m)
1049270	Tubing Insulator (for rainout and noise reduction)
1007800	High-Temperature Tubing, Beige (1,83 m)
1006121	Reusable Patient Tubing with Pressure Line (1,83 m)



622038



1032907



1049270



1007800

## Reusable Valves and Swivels

Part Number	Reusable Valves and Swivels
302312	Plateau Exhalation Valve (PEV)
332113	Whisper Swivel 2
622148	Outlet Swivel with Pressure Tubing



302312



332113



622148

## Other Accessories

Part Number	Other Accessories
312710	O <sub>2</sub> Enrichment Port
302418	Pressure Valve
1014047	Bacteria Filter
FC06566	Mouthpiece, angled, 22 mm



312710



302418



1014047



FC06566

# 3 steps to successful mask fitting


## SIZE SELECTION

1


### NASAL MASK SIZING

#### Landmarks

- a. Sides of nostrils
- b. Bridge of nose zone
- c. Below the nose tip, above the lip

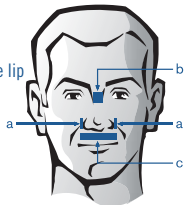


### USE THE SIZING GAUGE TO SELECT THE RIGHT SIZE

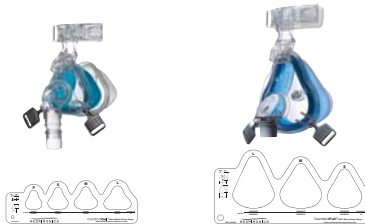

### FULL FACE MASK SIZING

#### Landmarks

- a. Sides of mouth
- b. Bridge of nose zone
- c. Below the lower lip

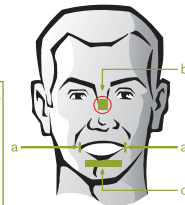

*Make sure the mouth is slightly open when checking size and the top of the cushion is not too high on the bridge of the nose area.*

## MASK FITTING

2

### Common Mistake

Stability Selector adjusted too high / Bottom strap too tight = Leaks into eyes, top lip pressure / irritation



### How to Adjust ?

1. Click and slide the adjuster downwards
2. Loosen the bottom strap of headgear



One click on the Stability Selector may make the difference.  
Always move the Stability Selector in the opposite direction of the leak

# OK



### Common Mistake

Stability Selector adjusted too low / Bottom strap too tight = Leaks around the mouth, forehead and bridge nose irritation / pressure points



### How to Adjust ?

1. Click and slide the adjuster upwards
2. Loosen top strap of headgear




**Correct position =**  
**Mask parallel to the face**  
**Do not overtighten the headgear**

## FINAL ADJUSTMENTS AND CLEANING INSTRUCTIONS

3

Check the adjustments of the mask in lying position with the right pressure levels.

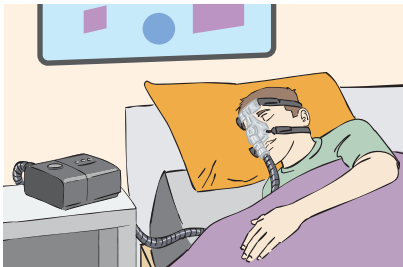

### Cleaning instructions

#### Mask and Tubing (Wash Daily)

- Hand wash the mask in warm water with mild dishwashing detergent.
- Rinse thoroughly and air dry completely before use.
- Wash face carefully before each use.
- Inspect the mask after cleaning; replace the mask and nasal interface cushion if damaged or worn.
- Rinse the tubing under shower and hang to dry.



#### Headgear (Wash Weekly)

- Handwash the headgear with mild detergent and warm water.
- Rinse thoroughly. Pat with a towel to absorb excess moisture, then hang to dry.







⚠ DO NOT use bleach, alcohol, any strong household cleaners, or cleaning solutions with alcohol, conditioners or moisturizers.

### ADDITIONAL FITTING TIPS

- Check that the headgear is not overtightened: allow 1 finger/hand space between the face and each of the straps
- Reposition the mask if necessary, by gently pulling the cushion away from the nose and then back to the initial fit



# Which device for which patient?

		SLEEP THERAPY			HOME N.I.V.		
							
		REMstar® Pro MSeries with C-Flex™	REMstar® Auto / A-Flex MSeries with C-Flex™ and A-Flex™	BiPAP® Auto MSeries with Bi-Flex®	BiPAP® autoSV™	BiPAP® Harmony®	BiPAP® Synchrony® with AVAPS™
TARGET PATIENTS		OSA	OSA	OSA (Patients unable to tolerate and/or non-responsive to CPAP)	Complex Breathing Pattern such as: CSR, CSA, Mixed Apnea and Complex Apneas	CPD Overlap syndrome	CPD, Overlap Syndrome, Restrictive respiratory syndromes: <small>obesity-hypoventilation, neuromuscular disorders, chest deformities, etc.</small>
	MODES	Fixed CPAP	Fixed CPAP Auto CPAP	BiPAP S Auto BiPAP S	Setup possible as: CPAP, CPAP + Servo Ventilation, BiPAP S, BiPAP ST and BiPAP + Servo Ventilation	CPAP, S [spontaneous], S/T [spontaneous / timed]	CPAP, S, S/T, PC [pressure control], T[timed]

Humidifier available in option 

**RESPIRONICS®**

**Envisioning tomorrow. Improving today.™**

Respironics Inc.: +1-724-387-4000

Respironics Europe, Africa, Middle East: +33-1-47-52-30-00

Respironics Deutschland: +49-8152-93-06-0

Respironics Switzerland: +41-62-745-1750

Respironics France: +33-2-51-89-36-00

Respironics Italy: +39-03-62-63-43-1

Respironics UK: +44-800-1300-845

Respironics Asia Pacific: +852-3194-2280

Respironics Australia: +61-2-9666-4444

[www.respironics.eu](http://www.respironics.eu) - [www.respironics.com](http://www.respironics.com)

CAUTION: U.S. federal law restricts these devices to sale by or on order of a physician. Not all products available in all countries, please check with your local Respironics representative for availability. Respironics, ComfortGel, Profilite, Comfortusion, ComfortGel Full, Comfortite 2 and Optilife are trademarks of Respironics, Inc. and its affiliates. © 2008 Respironics, Inc. and its affiliates. All rights reserved. RIT.RP1.011 E